


Control Panel Guide

Product Code- FC-PHR0046

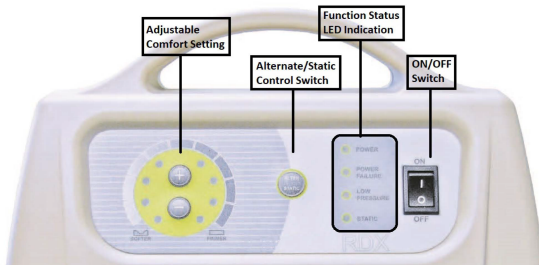

| | |
|--|---|
| <p>Comfort Setting - Choose from 1 of 8 selectable levels of air pressure for support of the resident. Manufacturer recommends using a hand check to select proper pressure setting for an individual resident.</p> | <p>LED Indicator - POWER light indicates the pump is in operate mode POWER FAILURE and LOW PRESSURE alarm indicators STATIC light indicates the pump is in static therapy mode</p> |
| <p>Therapy Mode - The mattress defaults to 2 in 1 Alternate mode with a 10 minute cycle time. Press and hold the ALTER/STATIC button for 3 seconds to toggle between Alternate and Static Mode therapy.</p> | <p>Main Power - ON/OFF switch will operate or shut down the pump.</p> |

Enhance RDX Suggested Weight Guideline

| Level | Suggested weight range (KG) | Suggested weight range (LB) |
|-------|-----------------------------|-----------------------------|
| P1 | 40-55kgs | 88-121lbs |
| P2 | 55-70kgs | 121-154lbs |
| P3 | 70-85kgs | 154-187lbs |
| P4 | 85-100kgs | 187-220lbs |
| P5 | 100-115kgs | 220-253lbs |
| P6 | 115-130kgs | 253-286lbs |
| P7 | 130-145kgs | 286-319lbs |
| P8 | 145-159kgs | 319-350lbs |

Please note that the above charts are just a guideline, not a rule. Because of different body types, shapes and height we recommend using the hand check method to ensure that the pressure setting is optimal and that the patient isn't bottoming out.

Operational Guidelines

- Carefully plug power cord of Control Unit into a properly grounded power source.
- Use the ON/OFF switch to turn the Control Unit to operate. The POWER LED will illuminate, the pump pressure dial blinks to the Max setting and the mattress will start to inflate. DO NOT press any buttons on the Control Panel until pump pressure dial defaults to solid lights and mattress is fully inflated.
- To determine the proper Comfort Setting, the manufacturer recommends performing a hand check.
- The mattress defaults to ALTERNATE mode. This setting provides 2 in 1 alternating pressure therapy for treatment and prevention of pressure ulcers. The cells inflate and deflate in a 2:1 cycle, meaning 50% of the body is always supported by air at any one time. The default cycle time is 10 minutes.
- To use STATIC therapy mode, press and hold the the ALTER/STATIC button for 3 seconds. The STATIC LED indicator will blink then become solid, letting the user know the mattress is in STATIC therapy. In STATIC therapy all 18 air cells are the same pressure.
- If the Low Pressure Alarm is alarming, check the following items — a. Make sure the connector for the mattress is hooked properly to the Enhance RDX pump. b. Check the CPR connector at the head of the bed and ensure that it is in the “Closed” position. c. Inspect the inside of the mattress and make sure all 18 cells are properly connected to the manifold, and no cells are punctured or torn.
- PLEASE REMEMBER TO CLEAN THE FILTER ON THE BACK OF THE ENHANCE RDX AT LEAST EVERY 60 DAYS. The white filter that is housed in the filter receptacle can be rinsed under running water and set to dry for 15 minutes. If you notice any dust clogs on the filter opening, use a vacuum to remove.
- CPR Deflation: Open the Red CPR valve located at the head of the mattress and disconnect the air hose connectors from the controller to quick release the air when in an emergency where CPR is to be performed.

Cleaning

The mattress should be cleaned on the bed weekly using a damp soft cloth and mild detergent. If top cover or base cover becomes grossly soiled, put on clean gloves, plastic gown and eye protection before removing top and base covers and dispose of according to standard hospital procedures for contaminated waste and replace with clean covers.

Covers can be washed and thermally disinfected in a washing machine. Never use phenol based cleaning solutions.