


## Control Panel Guide

Product Code- FC-PHR0032

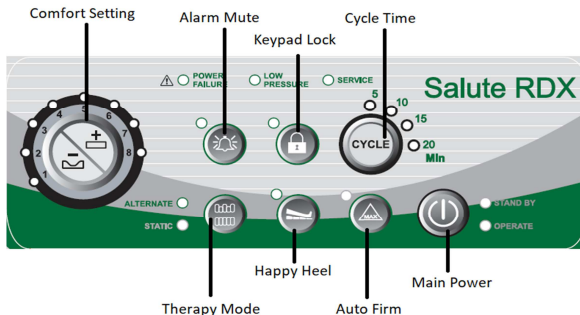

<p><b>Comfort Setting</b> - Choose from 1 of 8 selectable levels of air pressure for support of the resident. Manufacturer recommends using a hand check to select proper pressure setting for an individual resident.</p>	<p><b>Main Power</b> - After main power switch is turned on (side of pump) press and hold this button to move pump from Standby Mode to Operate Mode which activates pump.</p>
<p><b>Alarm Mute</b> - Press to mute on/off the audible alarms on pump.</p>	<p><b>Auto Firm</b> - Firms up the mattress to allow for easier egress, ingress or patient transfer.</p>
<p><b>Keypad Lock</b> - Press and hold for 5 seconds to lock/unlock all controls.</p>	<p><b>Happy Heel™</b> - Press to reduce pressure in the heel section by 25% of the rest of mattress to aid with residents' fragile heels.</p>
<p><b>Cycle Time</b> - When mattress is in Alternate Mode, this feature allows you to select the duration of alternation in 5, 10, 15 or 20 minute increments.</p>	<p><b>Therapy Mode</b> - Press to toggle between Alternate and Static Modes on mattress. When in Alternate Mode it will provide 3 in 1 alternating pressure therapy.</p>

## Salute RDX Suggested Weight Guideline

Level	*Suggested weight range (KG)	*Suggested weight range (LB)
P1	40-65kgs	88-145lbs
P2	55-83kgs	121-184lbs
P3	75-110kgs	163-244lbs
P4	100-133kgs	220-294lbs
P5	120-159kgs	265-352lbs
P6	150-182kgs	331-401lbs
P7	180-200kgs	397-441lbs
P8	200-227kgs	441-500lbs

*\*Please note that the above charts are just a guideline, not a rule. Because of different body types, shapes and height we recommend using the hand check method to ensure that the pressure setting is optimal and that the patient isn't bottoming out.*

## Operational Guidelines

- Carefully plug power cord of Control Unit into a properly grounded power source. Turn on the master mechanical power switch on the right side panel. The STANDBY LED will illuminate on the keypad.
- Press the Main Power Button. The OPERATE LED will illuminate and the airflow will commence.
- To determine proper Comfort Setting, the manufacturer recommends performing a hand check.
- The mattress defaults in STATIC mode. Press the THERAPY MODE button for ALTERNATE mode. This setting provides 3 in 1 alternating pressure therapy for treatment and prevention of pressure ulcers. The cells inflate and deflate in a 3:1 cycle, meaning 2/3rds of the body is always supported by air at any one time.
- The CYCLE button controls the alternating pressure cycle time in the ALTERNATE mode. CYCLE is selectable from 5, 10, 15 AND 20 minutes. A CYCLE marks the time each 1/3 set of bladders are deflated.
- If the Low Pressure Alarm is alarming, check the following items — a. Make sure the connector for the mattress is hooked properly to the Salute RDX pump. You should hear 2 audible clicks when inserting the mattress connector to the pump. b. Check the CPR connector at the head of the bed and ensure that it is in the “Closed” position. c. Inspect the inside of the mattress and make sure all 18 cells are properly connected to the manifold, and no cells are punctured or torn.
- PLEASE REMEMBER TO CLEAN THE FILTER ON THE BACK OF THE SALUTE RDX AT LEAST EVERY 60 DAYS. The white filter that is housed in the filter receptacle can be rinsed under running water and set to dry for 15 minutes. If you notice any dust clogs on the filter opening, use a vacuum to remove
- CPR Deflation: Open the Red CPR valve located at the head of the mattress and disconnect the air hose connectors from the controller to quick release the air when in an emergency where CPR is to be performed.
- If the SERVICE LED is illuminated, please call your local mattress provider and they can perform the necessary maintenance on the unit.

## Cleaning

The mattress should be cleaned on the bed weekly using a damp soft cloth and mild detergent. If top cover or base cover becomes grossly soiled, put on clean gloves, plastic gown and eye protection before removing top and base covers and dispose of according to standard hospital procedures for contaminated waste and replace with clean covers.

Covers can be washed and thermally disinfected in a washing machine. Never use phenol based cleaning solutions.